





Assembly Instructions For Your Face Shield

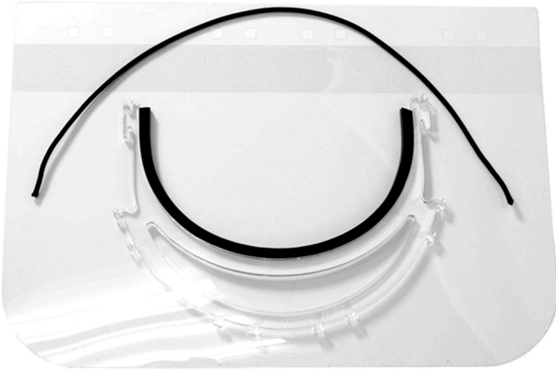

Place parts on table



Take shield sheet and place the first 2 openings over the first 2 prongs

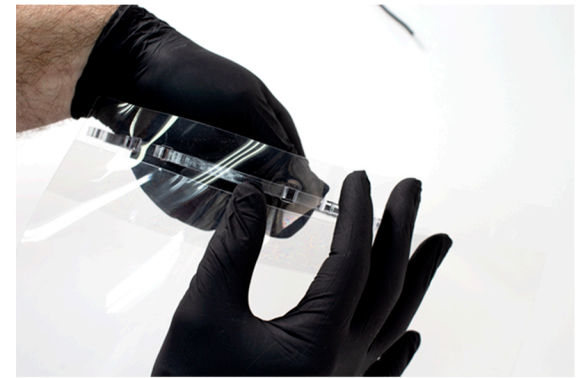

Continue popping the next 2 openings onto the next prongs



Continue popping the next 2 openings onto the next prongs



Finish up by popping the last 2 openings onto the last prongs



Now that all openings are popped onto all prongs, prepare for strap




Take the first end of the strap



Make a loop into it



Put the end through the loop

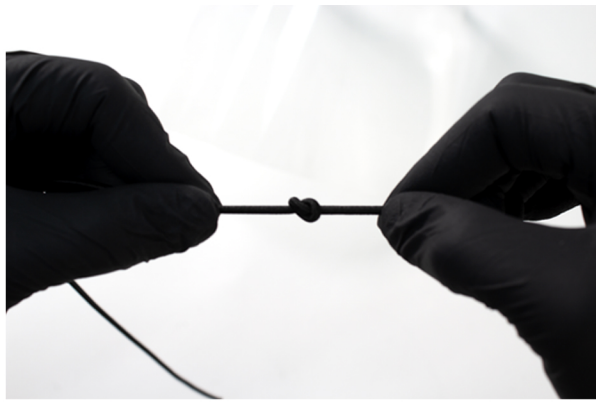

Pull tight



Take the second end of the strap



Make a loop into it



Put the end through the loop



Pull tight



Hook one side of strap (knot top)



Extend to other side of strap



Hook other side of strap (knot top)



Ready to wear!